



*ahead of the curve*

**Quart**  
HEALTHCARE

## **Quart Healthcare Low Air Loss Mattress Cleaning Checklist**

### **COVER**

- Wipe cover
- Remove the cover and spray with Accel-TB
- Wait to allow 5 minute contact time
- Wash cover in hot water wash with regular laundry detergent
- Hang to dry

**\*\* For DynaLAL, mesh middle section must be removed and washed with covers. Mesh can be removed by disconnecting snaps on opposite side of air connectors**

### **MATTRESS**

- Spray entire mattress with Accel-TB
- 5 minute contact time
- Wipe with clean cloth.
- Once the mesh and cover are dry, reassemble
- Bag and tag

### **CLEANING THE PUMP**

- Wipe pump
- Spray the pump with **Accel-TB**
- 5 minute contact time
- Rinse filter
- Squeeze to dry